

Medison Volume files:

### STEP 1

Selecting the image and convert it to a Volume

1. Select the required 3D image.
2. You need to save the chosen image to a version with the Volume data included
3. If images are saved with volume data, they can be converted to new 3D rendering images with SONOVIEW.
4. Select VOLUME and than DEFAULT save to SONOVIEW



### STEP 2

Exporting the image to TIFF and VOLUME file

1. Go to SONOVIEW
2. Select the 3D image that was just saved with Volume data
3. Click the Export Image icon on the screen. The Image Export window will be displayed.
4. Specify the various parameters, such as directory, drive, and filename
5. Choose File format TIFF
6. Export option select only 3D VOLUME DATA. Deselect the other two options.
7. Click the Export button to start a transfer. You will get 3 files send us all 3 files.

